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## Tumle Cusion, Tumle Dental and MobiBed

User Manual – Short Version

Applies to products with article numbers starting with 0305-, 0316-, 0317-, 0319-, and ending with -000, -120, -010, -015, -020, as well as other products. The article number is indicated on the product label.

- For cleaning, wipe the inner cushion with commonly used cleaning agents, such as dish soap.
- The cushion must not be submerged in water.
- Disinfection of the inner cushion and hygiene cover should be done with commonly used disinfectants containing surfactants. It is important to work (rub) the surface for the best effect.
- Always check the disinfectant's handling recommendations.
- Be careful not to use disinfectants or cleaning agents with a pH value above 11.4.
- Do not use solvents or products containing solvents.
- **Note!** The beads in the filling melt at 71°C.

**Cotton covers** can be machine washed at up to 60°C and tumble dried. Washing above 60°C may cause colour fading, and the fabric may shrink by up to 5%. Wash the cotton cover with similar colours.

**The hygiene cover** is washable and can be tumble dried at up to 95°C and used with all industrial detergents. Do NOT use fabric softeners. The hygiene cover has a coating that resists moisture. When machine washing, it is recommended to wash with textiles without this coating to ensure proper water flow.

**EKO covers** can be machine washed at up to 60°C and tumble dried on low heat. Washing above 60°C may cause colour fading, and the fabric may shrink by up to 5%. Wash with similar colours.

The cushion and accessories can be disposed of as regular household waste. A more detailed user manual is available at [www.cobirehab.com](http://www.cobirehab.com).

C/o Helio, Hornsbruksgatan 23B | 117 34 Stockholm | Telefon 08-702 97 74 | [cobirehab.se](http://cobirehab.se)

## Tumle Floating Cushions and Hygiene Cushions

User Manual – Short Version

This user manual applies to Tumle floating cushions and hygiene cushions with article number 3002-XXX-000.

Tumle floating cushions do not meet the requirements for lifesaving equipment and must not be used as a substitute for a life jacket or life vest. Responsibility for the swimmer's safety, risk of drowning, or other injuries always rests with the assisting personnel.

**When used** in hygiene situations, it is recommended that the cushion is assigned to a specific individual. Avoid exposing the cushions to sharp edges or pointed objects. If the cushions are punctured, causing the filling to escape, they will lose their buoyancy and positioning capabilities and must be discarded. Regularly inspect the cushions for damage. Avoid using the cushions outdoors in lakes or seawater. Contaminants can cause mould. Never use bath oils with Tumle floating cushions/hygiene cushions.

**After use** the cushion should be thoroughly rinsed with water and placed in an airy position with the mesh side facing down to allow water to drain easily. Then, let the cushion drip dry. Do not stack cushions on top of each other. If the cushions need to be packed for transport, ensure they are completely dry, as there is a risk of mould otherwise. It is preferable to store the cushions hanging.

- If necessary, the cushions can be machine washed at 60°C. A laundry bag must be used.
- Take care not to use disinfectants or cleaning agents with a pH value above 11.4.
- Do NOT use fabric softener, dry cleaning, solvents, or bleach.
- The cushions can be dried in a drying cabinet at a maximum of 40°C.
- **Note!** The beads in the filling melt at 71°C.

The cushion and accessories can be disposed of as regular household waste. A more detailed user manual can be found at [www.cobirehab.com](http://www.cobirehab.com).

C/o Helio, Hornsbruksgatan 23B | 117 34 Stockholm | Telefon 08-702 97 74 | [cobirehab.se](http://cobirehab.se)